
Cod. **0824**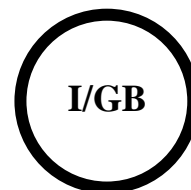

Via Don L. Sturzo,6 – 42021 Barco (RE) (ITALY)  
Codice Fiscale e Partita IVA: IT 01363800358  
Numero REA: 180051 C.C.I.A.A. di Reggio Emilia  
Tel. 0039 (0) 522 875159 Fax 0039 (0) 522 875579  
e-mail: reverberisrl@gmail.com

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SYMBOLS AFFIXED TO THE MACHINE		<b>WARNING: IDENTIFIES A HEAT SOURCE. A PART COULD BE DANGEROUSLY HOT</b>
		<b>WARNING: IDENTIFIES A GENERIC DANGER. THERE COULD BE A POTENTIALLY DANGEROUS CONDITION</b>
		<b>IDENTIFIES THE GROUND CONDUCTOR</b>

	<ul style="list-style-type: none"> <li>• This symbol it means the product is covered by the European Directive 2002/96/EC.</li> <li>• Please inform yourself about the local separate collection system for electrical products.</li> <li>• Please act according to your local rules and do not dispose of your old products with your normal household waste. The correct disposal of your old product will help prevent potential negative consequences for the environment and human health.</li> </ul>
---	--

## 1. WARNINGS

Do not load wet materials into the sterilization chamber of the unit to avoid the risk of short circuits.

## 2 INTRODUCTION

### 2.1. Expected use of materials to sterilize

The equipment is used to sterilize surgical instruments in the medical – paramedical sector.

The materials to be sterilized must **withstand** operating temperatures ranging from 0° to 200° C. **Do not load toxic materials into the sterilizer.**

### 2.2 Manufacturer's responsibility

The equipment is manufactured in compliance with European regulations.

The use of this equipment is restricted to the functions for which it was originally designed and manufactured. The user shall be responsible in case of any possible risks and dangers caused by an improper use of the sterilizer.

## 3. INSTALLATION

Carefully remove the sterilizer from its packing and make sure there are no sign of damage. Make sure earthing is correctly made. Put sterilizer on a flat surface, to facilitate aeration. Always respect a distance of at least 10 cm from any wall or other equipment, to allow suitable ventilation.

Do not put objects or other items in the vicinity of the door, to prevent them from hindering opening of same.

Connect the sterilizer to a nearby and easily accessible plug. Press the main switch setting it to ON and make sure the sterilizer switches on.

## 4. CHARACTERISTICS

### 4.1. Technical data

Weight : 13kg

Outer dimensions: Width cm 44 – Height cm 38 – Depth cm 23

Inner dimensions: Width cm 37,5 – Height cm 20 – Depth cm 23

Absorption: 620 watt

Power supply: 230V – 50Hz

### 4.2. Environmental working conditions

-Temperature 5° - 40°

-Moisture ( with max. T° 31°) 80%

-Moisture (with max T° 40°) 50%

### 4.3. Recommended times and temperatures

The temperatures that can be programmed depend on the specific characteristics stated by the manufacturer and/or on the instrument to be sterilized. The recommended temperatures for dry sterilization range from 150° to 180°. **IMPORTANT: the personnel using the sterilizer must respect above-mentioned temperatures.**

**Maximum programmable temperature: 200°C**

**The recommended sterilization times range from a minimum of 30 minutes to a maximum of 45/60 minutes.**

Maximum programmable time: 120 minutes. The sterilizer can also work indefinitely ( by turning the thermostat knob counter clockwise, as indicated on the nameplate placed on the sterilizer).

## 5. OPERATING MODES

- 1) Carefully clean and wipe the instruments to sterilize
- 2) Turn the main switch on
- 3) **To programme temperature:** Turn the thermostat knob and set the pointer on the temperature you wish to reach.
- 4) Turn the timer knob **clickwise** and set the pointer on time needed ( max. 120 minutes).

After completion of these operation, the two-colour pilot lamps will show you the operating status of the sterilizer ( green pilot lamp= operation, yellow pilot lamp= thermostat). Before stabilizing on the temperature programmed, the yellow pilot lamp will blink every now and then ( to indicate that temperature is rising ).

If you want to disconnect the timer and make the sterilizer operate indefinitely, turn the timer knob **counter clock wise** and set it to “manual” . For a correct use of the machine, we recommend not to overload the trays, to enable a good heat circulation.

To control temperature the sterilizer is equipped with a thermometer visualizing the programmed temperature.

## 6. TRANSPORT AND STORAGE CONDITIONS

The sterilizer is equipped with packing requiring normal transport precautions necessary for fragile materials, as indicated on the packing container itself. Make sure you retain the packaging materials for future shipment or should the machine need repairing. In case of storage, put the sterilizer in a moisture-free area to protect electrical parts.

## 7. MAINTENANCE AND CLEANING

Any maintenance must be executed after disconnecting the feeding cable and when the machine is completely cold.

Use a cloth soaked in alcohol to clean periodically the interior of the sterilizer chamber. **Avoid low or high pressure water jets.** Do not allow water to flow inside the machine to avoid the risk of short circuits.


Make sure periodically that the feeding cable is in a good condition.

In case of improper functioning, please contact the manufacturer.


In case of repairs, please send the sterilizer to the manufacturer in its original packing container.

**Control temperature on a periodic/yearly basis using the calibrated thermometer supplied by the manufacturer.**

## 8. WIRE DIAGRAM

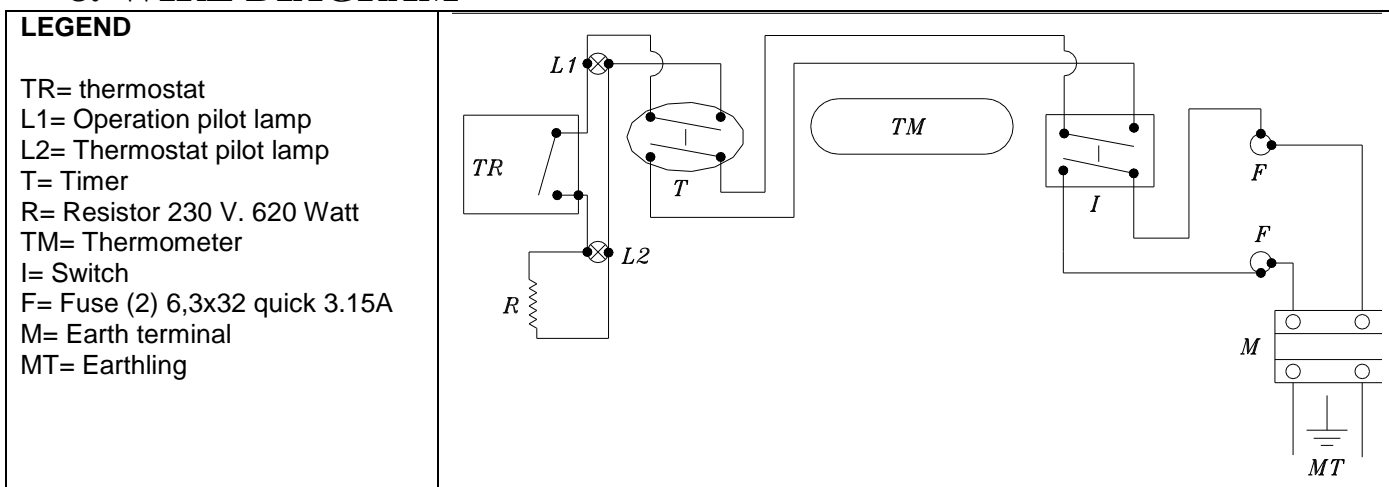## 9. CONTROL PANEL
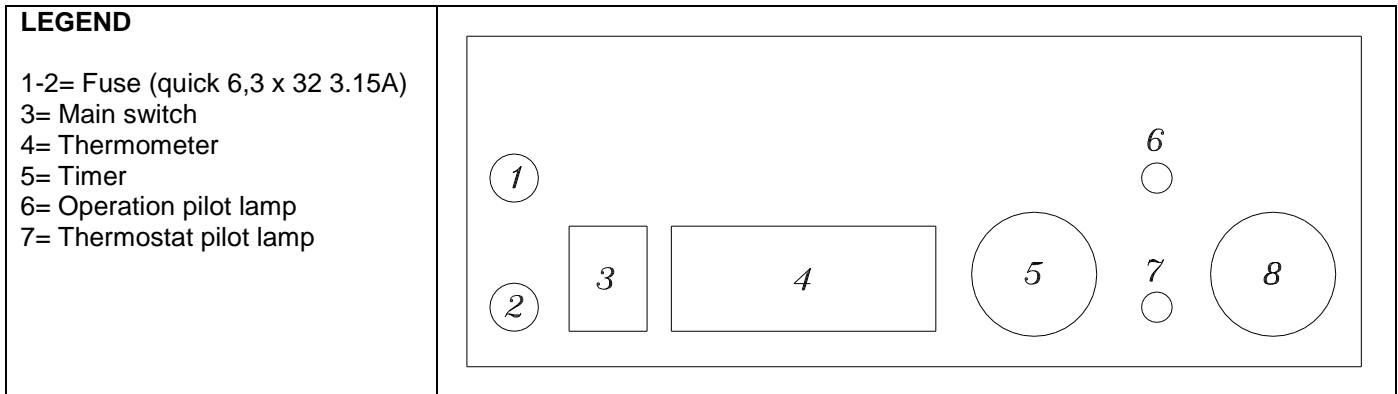
## 10. SAFETY RULES AND PRECAUTION

When operating the sterilizer always wear protective gloves to avoid possible burns (a **precautionary nameplate** “**WARNING: DANGER OF BURNS**” can be found on the outer frame of the sterilizer)

**PAY ATTENTION NOT TO EXTRACT THE TRAYS TOO MUCH WHEN TAKING THE INSTRUMENTS OUT OF THE STERILIZER; DANGER OF BURNS – WEAR PROTECTIVE GLOVES**

The sterilizer must be used only by an expressly authorized personnel, informed about the “Sterilization techniques”, the materials that can be sterilized and the recommended times and temperatures to follow.

This equipment must be used uniquely for the purpose for which it was originally designed and manufactured, i.e. sterilization of surgical instruments to be used in medical and paramedical field.

All precautions must be **absolutely** taken to avoid an **improper** use of the machine by **unauthorized** people. Modification, removal or disconnection of existing thermal and electric protections is strongly forbidden. The operator must not perform any repairs on the machine.

Always disconnect the sterilizer during cleaning and maintenance of the same. Moreover it must always be completely cold.

When sterilizing surgical instruments ( such as scalpels or pointed instruments) safety shoes should be preferably worn.

Check periodically that the feeding cable is not damaged.

# WARRANTY CERTIFICATE

This document certifies that the appliance has been correctly manufactured.

The appliance will be repaired if it operates in a faulty way during the warranty period of 24 months owing to manufacturing defects, the manufacturer's judgement over such matters being final.

The sales invoice or consignment note shall bear witness in the event of disputes.

The electrical, electronic parts and nuts and bolts are guaranteed for a period of 12 months.

The warranty covers the sole replacement or repair of those parts recognized as possessing manufacturing defects, travel and call costs excluded.

Replaced parts shall be returned free port. The ordering party shall be charged for any part that is not returned.

All appliances returned for repairs shall only be sent to us free port after authorization and shall be adequately packed, with the sender's name, address and claimed defect clearly indicated.

The appliance remains the property of the seller until it has been fully paid for by the purchaser.

## **THE WARRANTY SHALL BECOME VOID:**

1. If the appliance has been repaired, modified or tampered with by the purchaser or by unauthorized third parties.
2. If the purchaser fails to contact the seller or authorized technical center.
3. 18 months from the date of our consignment note.
4. If the customer delays or suspends payments.
5. If the appliance has been damaged by exposure to fire, spilt liquids, natural disasters, has been dropped or damaged by causes that are not ascribable to manufacturing defects.
6. When the warranty form has not been duly filled out, signed and returned to the manufacturer within the established time.
7. If the customer uses the appliance improperly, fails to carry out the routine maintenance operations or services the appliance in a neglectful way.